THUASNE

SPORT

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Conserver cette notice.

Indications

- Mild sprains of the lateral ligaments of the knee.
- Chronic knee instability (cruciate ligament rupture).
- Resumption of sporting activities following mild knee sprains and injuries (including surgery).
- Painful chronic knee conditions.

Properties and specific characteristics of the product:

- The elastic knit exerts uniform pressure on the joint and improves proprioception (the joint's sensory perception).
- Comfort zone over the kneecap and the crease behind the knee.
- Flexible side boning ensuring good support.
- Pull tabs for easier fitting.
- Anatomical shape.

Fitting instructions

- Pull the product on (the logo indicates the top and front of the product).
- Make sure the kneecap is properly centred in the zone provided for this purpose.

Precautions

- Do not apply directly to broken skin.
- Always use the right size for your morphology.
- In the event of discomfort or pain, consult a health professional.
- Do not use for any indications other than those indicated above.
- Store at room temperature, preferably in original packaging.
- For hygiene and performance reasons, reuse of this device by another person is not recommended.

Washing instructions

- See product label.
- Do not use detergents, fabric softeners or aggressive products (products containing chlorine).
- Squeeze out excess water.
- Dry away from heat sources.

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Thuasne Scandinavia AB
Thuasne Cervitex Ltd
Thuasne UK Ltd
Medical Devices Co., Ltd.**