THUASNE

Bandage rotulien

Patellar bandage

Patellasehnenbandage

Patella bandage

Bendaggio rotuleo

Cincha rotuliana

Liga rotuliana

Knæbind

Patellan tukiside

Knäskålsbandage

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(en)

Properties :

- The patellar bandage applies local pressure to the patellar tendon and supports the kneecap, thereby absorbing vibrations.
- Enhanced proprioceptive effect thanks to the combination of silicone with dimples, which creates a massage effect and a fabric band, which provides even compression.
- One size fits all.

Indications :

- Absorption of vibrations.
- Proprioception.
- Prevention of tendinitis of the patellar tendon.

Contraindication :

- Do not apply the product in direct contact with broken skin.

Side effects:

No side effects are as yet known for correct use.

Fitting instructions :

- First unfasten the tightening strap.
- The patellar bandage must be placed over the sub-patellar tendon, remembering that the patellar tendon must be positioned at the centre of the silicone insert.
- Next, re-fasten the strap around the knee.

Precautions for use :

- Do not tighten the patellar bandage excessively to avoid any tourniquet effect.
- In the event of discomfort or pain, consult a health professional.
- Do not use for indications other than those indicated above.
- For hygiene and performance reasons, do not re-use the product for another patient.

Washing instructions :

- See product label.
- Do not use detergents, fabric softeners or aggressive products (products containing chlorine).
- Squeeze out excess water.
- Dry away from heat sources.

Composition :

Nylon - Polyethylene - Silicone - Cotton

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Thuasne Deutschland GmbH
Thuasne Benelux B.V.
Thuasne Scandinavia AB
Thuasne Cervitex Ltd
Thuasne UK Ltd
Thuasne (Shanghai) Medical Devices Co., Ltd.