
THUASNE

Ligastrap® Genu

- (fr)** Genouillère ligamentaire à sanglage fonctionnel
- (en)** Ligament knee brace with functional strapping
- (de)** Knieführungsbandage mit funktionellen Zugbändern
- (nl)** Ligamentair knieverband met functionele banden
- (it)** Ginocchiera con fasciatura funzionale per legamenti
- (es)** Rodillera para ligamentos con cinchas funcionales
- (pt)** Joelheira ligamentar com tiras funcionais
- (da)** Knæstøttebandage med funktionelle elastiske bånd
- (fi)** Polvituki nivelsiteiden hoitoon, toiminnallisuutta edistävät tukisiteet
- (sv)** Knäortos med funktionellt remsystem
- (el)** Επιγονατίδα τενόντων με λειτουργικό δέσιμο
- (cs)** Ortéza pro kolenní vazy s korekčními pásky
- (pl)** Orteza stawu kolanowego stabilizująca z funkcjonalnymi pasami dociągowymi
- (lv)** Ceļa saišu pārsējs ar funkcionālo saiti
- (lt)** Ligamentinis kelio įtvartas su funkcinė dirželių sistema
- (et)** Funktsionaalse rihmasüsteemiga põlveortoos
- (sl)** Kolenska opornica s funkcionalnimi trakovi
- (sk)** Strapovacia kolenná ortéza (na podporu vāzov)
- (hu)** Ligament térdrögzítő funkcionális pántokkal
- (bg)** Ортеза за връзките на коляното с функционални ленти
- (ro)** Orteză de genunchi ligamentară
- (ru)** Ортез коленный лигаментарный с функциональными ремнями
- (zh)** 带有功能性搭扣的韧带护膝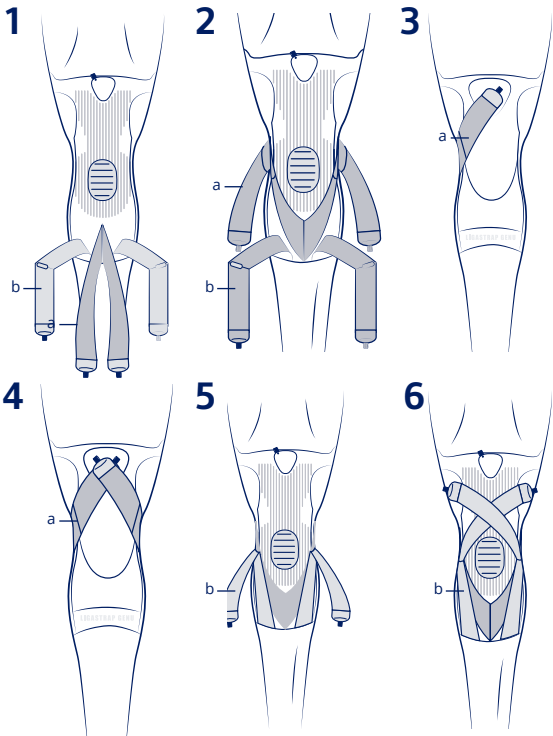

	32 - 34 cm	1
	35 - 37 cm	2
	38 - 41 cm	3
	42 - 45 cm	4
	46 - 49 cm	5
	50 - 53 cm	6

- | | | |
|---------------------------|------------------|-----------------------------|
| (!) Proprioception | (T) Compression | (T) Stabilisation |
| (!) Proprioception | (T) Compression | (T) Stabilization |
| (!) Proprioception | (T) Kompression | (T) Stabilisierung |
| (!) Proprioceptie | (T) Compressie | (T) Stabilisatie |
| (!) Propriocezione | (T) Compressione | (T) Stabilizzazione |
| (!) Propriocepção | (T) Compressão | (T) Estabilização |
| (!) Propriocepção | (T) Kompresão | (T) Estabilizaçào |
| (!) Asento- ja liikeaisti | (T) Kompresio | (T) Stabilisering |
| (!) Proprioception | (T) Kompresion | (T) Stabilisering |
| (!) Πρόληψη | (T) Συμπίεση | (T) Σταθεροποίηση |
| (!) Propriocepce | (T) Kompresje | (T) Stabilizace |
| (!) Propriocepçja | (T) Kompresja | (T) Stabilizacija |
| (!) Propriocepçija | (T) Saspiešana | (T) Stabilizãcija |
| (!) Propriocepçia | (T) Kompresija | (T) Stabilizavimas |
| (!) Liigutustundlik | (T) Kompresioon | (T) Stabiliseeriv |
| (!) Propriocepçija | (T) Kompresija | (T) Stabilizãcija |
| (!) Propriocepçia | (T) Kompresia | (T) Stabilizãcia - fixovane |
| (!) Propriocepçió | (T) Kompreszió | (T) Stabilizãlás |
| (!) Προπριοϑεπτιβνση | (T) Κομπρεια | (T) Σταβιλιζαçia |
| (!) Propriocepçje | (T) Kompresie | (T) Stabilizare |
| (!) Προπριοϑεπçia | (T) Κομπρεια | (T) Σταβιλιζαçia |
| (!) 本体感受 | (T) 压力 | (T) 稳定 |

MD

- | | |
|-----------------------------|---|
| (T) Dispositif medical | (T) Un seul patient - à usage multiple |
| (T) Medical device | (T) Single Patient - multiple use |
| (T) Medizinprodukt | (T) Einzeler Patient - mehrfach anwendbar |
| (T) Medisch hulpmiddel | (T) Eén patiënt - meervoudig gebruik |
| (T) Dispositivo medico | (T) Singolo paziente - uso multiplo |
| (T) Producto sanitario | (T) Un solo paciente - uso multiple |
| (T) Dispositivo médico | (T) Paciente único - várias utilizaçoes |
| (T) Medicinsk udstyr | (T) Enkelt patient - flergangsbrug |
| (T) Lääkinällinen laite | (T) Potilaskohtainen - voidaan käyttää useita kertoja |
| (T) Medicinteknisk produkt | (T) En patient - flera användningar |
| (T) Ιατροτεχνολογικό προϊόν | (T) Πολλαπλή χρήση - σε έναν μόνο ασθενή |
| (T) Zdravotnický prostředek | (T) Jeden pacient - vícenásobné použití |
| (T) Wyrób medyczny | (T) Wielokrotne użycie u jednego pacjenta |
| (T) Mediciniska ierçce | (T) Viens pacients - vairãkkãrtēja lietošana |
| (T) Medicinos priemonė | (T) Vienas pacientas - daugkartinis naudojimas |
| (T) Meditsiiniseade | (T) Ühel patsiendil korduvalt kasutatav |
| (T) Medicinski pripomoček | (T) En bolnik - večkratna uporaba |
| (T) Zdravotnicka pomôçka | (T) Jeden pacient - viacnãsobné použitie |
| (T) Orvostechnikai eszköz | (T) Egyetlen beteg esetében többször újrahaználható |
| (T) Медицинско изделие | (T) Един пациент - многократна употреба |
| (T) Dispozitiv medical | (T) Un singur pacient - utilizare multiplã |
| (T) Медицинское изделие | (T) Многократное использование для одного пациента |
| (T) 医療器械 | (T) 一位患者 - 多次使用 |

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LIGAMENT KNEE BRACE (en) WITH FUNCTIONAL STRAPPING

Properties:

- Elastic fabric.
- Double functional strapping ensuring good lateral support.
- Flexible lateral reinforcement.
- Very light, comfortable knitted zone at the crease behind the knee.
- Partial front opening to make fitting easier.
- Anti-slip system to prevent the knee brace slipping.

Indications:

- Functional treatment of mild sprains.
- Resumption of occupational activities and sport after sprains.
- Chronic hyperlaxity.

Contraindication:

Do not apply the product in direct contact with broken skin.
Do not use in the event of thiuram allergy.

Side effects:

No side effects are as yet known for correct use.

Fitting instructions:

- Slip on the knee brace (partial opening at the front), making sure that the kneecap is properly positioned in the flexible knitted zone provided for this purpose.
- Pull the straps (a) and fix them at the back on the felt pad at the top of the knee brace (diagrams 1, 2, 3 and 4).
- Pull the side straps (b) up and fix them half way up on the self adhesive material. Cross them over and place them on the felt pad at the top of the knee brace, in front of where the previous straps were fixed (figures 5 and 6).
- For comfort during wear, make sure the straps are not creased.

Precautions:

Follow the advice of the professional who prescribed or supplied the product. Wearing this product, as with any compression product, could, in some persons, cause cutaneous reactions (redness, erythema). Should this occur, it is preferable to readjust or remove the product and ask the professional who delivered the product for advice. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient.

Washing instructions:

- See product label.
- Do not use detergents, fabric softeners or aggressive products (products containing chlorine).
- Squeeze out excess water.
- Dry away from heat sources.

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Thuasne Scandinavia AB
Thuasne Cervitex Ltd
Thuasne UK Ltd
Thuasne (Shanghai) Medical Devices Co., Ltd.